


VENTANA ALK (D5F3) CDx Assay
*Multiple targeted therapy options for
ALK+ NSCLC patients*

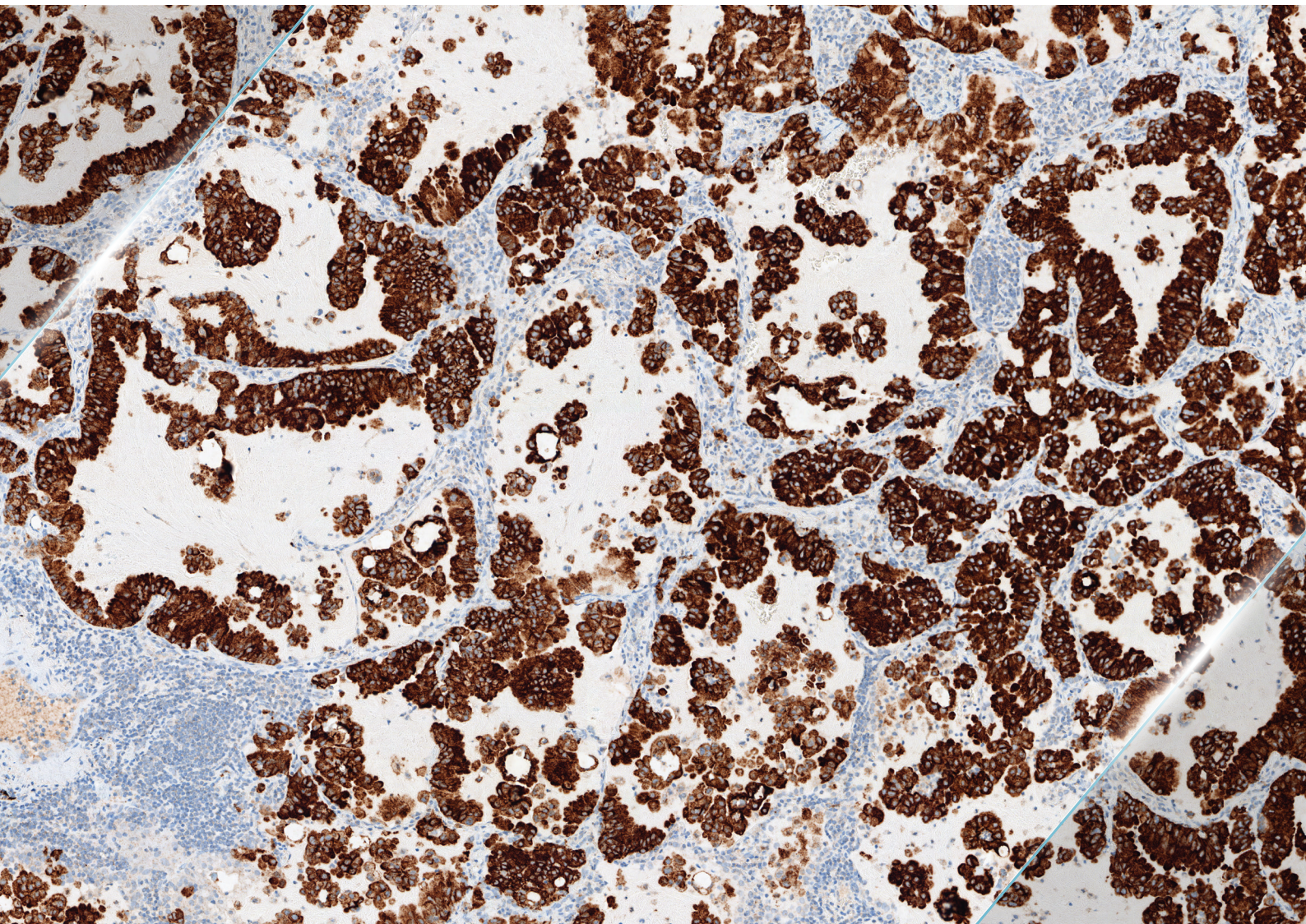

VENTANA ALK (D5F3) CDx ASSAY

S18 – 4341 A1 – 2 	S18 – 4341 A1 – 2 	S18 – 4341 A1 – 2 
		
+ +	+ +	+ +

VENTANA ALK (D5F3) CDx Assay* is intended for laboratory use in the detection of the anaplastic lymphoma kinase (ALK) protein in formalin-fixed, paraffin-embedded non-small cell lung carcinoma (NSCLC) tissue stained with the BenchMark series instruments.

*NSCLC tissue samples stained with the VENTANA ALK (D5F3) Assay and OptiView DAB Detection and Amp

Identify ALK+ NSCLC patients eligible for treatment with XALKORI®, ZYKADIA®, ALECENSA® or LORBRENA®



VENTANA ALK (D5F3) Assay stained with OptiView DAB IHC Detection and Amplification kits detects the ALK protein that is the target of therapy

Intended use

It is indicated as an aid in identifying patients eligible for treatment with XALKORI® (crizotinib), ZYKADIA® (ceritinib) or ALECENSA® (alectinib) or LORBRENA® (lorlatinib).

This product should be interpreted by a qualified pathologist in conjunction with histological examination, relevant clinical information and proper controls.



This antibody is intended for in vitro diagnostic (IVD) use.

*Abbreviated to VENTANA ALK (D5F3) Assay

ALK IHC value points

- Clinical guidelines recommend rapid turnaround for earlier targeted therapies¹
- According to the NCCN guidelines: FDA-approved IHC (ALK (D5F3) CDx Assay) can be utilized as a stand alone test, not requiring confirmation from FISH.³³
- More immediate treatment decisions can be made for advanced NSCLC patients by using the VENTANA ALK (D5F3) Assay
- VENTANA ALK (D5F3) Assay has comparable sensitivity and specificity relative to FISH³
- XALKORI (crizotinib), ZYKADIA (ceritinib) and ALECENSA (alectinib) and LORBRENA® (lorlatinib) are clinically effective and recommended for the treatment of ALK-positive patients^{2, 3, 4, 5}

About non-small lung cancer

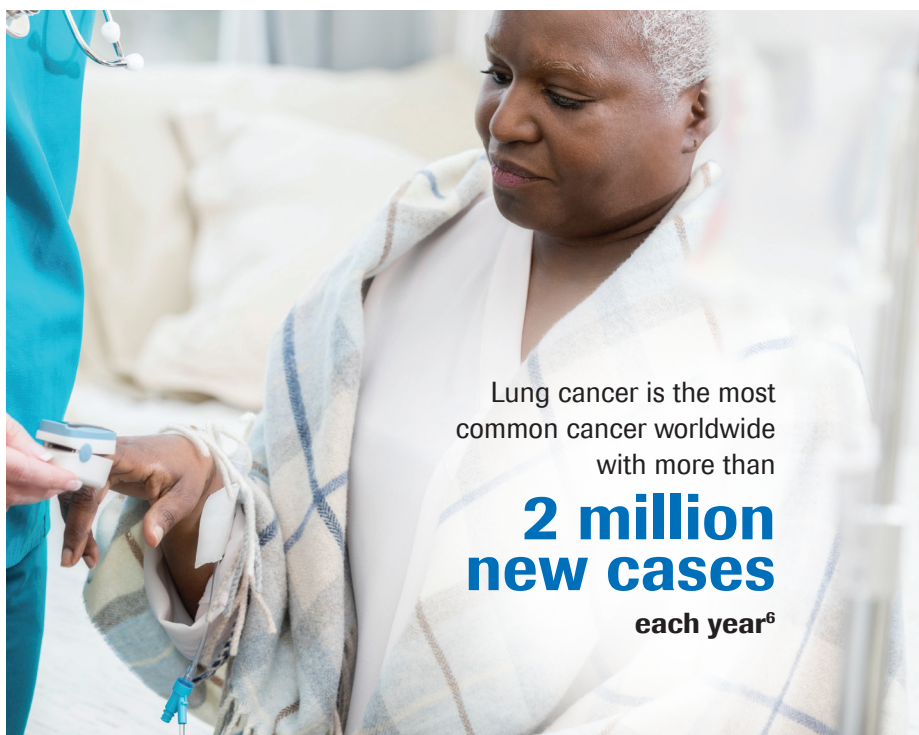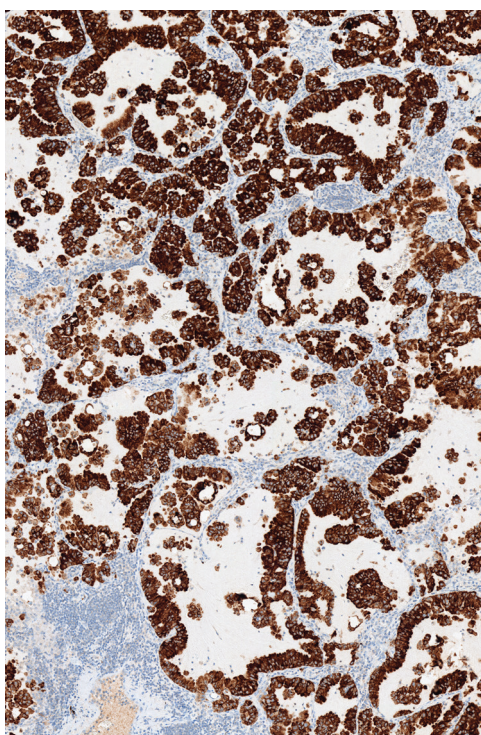

Lung cancer is the most common cancer worldwide with more than

2 million new cases

each year⁶

Over the past three decades, lung cancer has shown the least improvement in survival rates when compared with other cancers.⁷

Lung cancer has been the most common cancer in the world for several decades and remains the leading cause of cancer deaths worldwide.¹⁰ It is estimated to account for 12.9% of all new cancer cases and is responsible for nearly 1.59 million deaths annually worldwide, or approximately one in five cancer-related deaths.⁸ Although improvements have been made in diagnosis and therapy options, prognosis remains poor with low long-term survival rates for all stages.

Non-small cell lung cancer (NSCLC), one of the two major types of lung cancer, accounts for approximately 85% of all lung cancer

cases.⁶ In more than half of patients newly diagnosed with NSCLC, the disease has already metastasized, greatly decreasing the likelihood of survival. The five-year relative survival rate for NSCLC diagnosed as distant disease is 4.7%.¹⁷ The majority of patients with NSCLC present with inoperable, locally advanced disease (Stage IIIB) or metastatic disease (Stage IV), neither of which currently has any curative treatment options. On average, these patients die within a year of diagnosis. Improvement in the clinical outcome of lung cancer is likely to be achieved through better understanding of the molecular events that underlie its pathogenesis, identifying new biomarker targets and developing new treatment options.

Lung cancer is the leading cause of cancer death for men and women globally with more than


4,000 people dying every day¹⁰

ALK is considered one of the key oncogenic drivers in NSCLC



Lung cancer is the most prevalent form of cancer in the world. Each year, more than 2 million new cases are diagnosed. Lung cancer also has the highest mortality rate. Five-year survival rates are as low as 24%. Adenocarcinoma, a subset of NSCLC, is the most common, comprising approximately 40% of all lung disease.^{16, 17}

Five-year survival rate of patients after diagnosis with NSCLC is

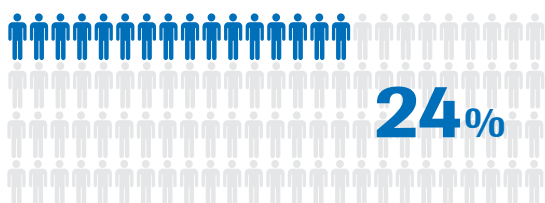

ALK mutation in lung cancer

Genetic mutations are known to play critical roles in the progression to metastatic lung disease. The majority of these mutations are found in adenocarcinoma of young non-smokers. ALK is considered one of the key oncogenic drivers in NSCLC. The ALK gene codes for a transmembrane glycoprotein with tyrosine kinase activity. In-frame rearrangements with the known fusion partners place the ALK kinase domain under the control of a different gene promoter. This fusion results in a chimeric protein (like EML4-ALK) with constitutive tyrosine kinase activity that has been demonstrated to play a key role in controlling cell proliferation. This unique protein is also a potential target for ALK-specific tyrosine kinase inhibitor (TKI) therapy.^{18, 8, 19}

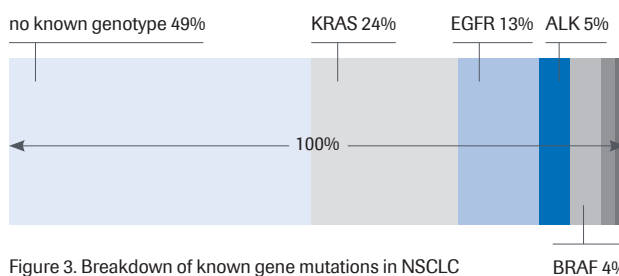


Figure 3. Breakdown of known gene mutations in NSCLC


BRAF 4%

Testing for lung cancer

Main types of lung cancer¹¹



Poor 5-year survival rate when compared to other cancers (U.S. only)¹²

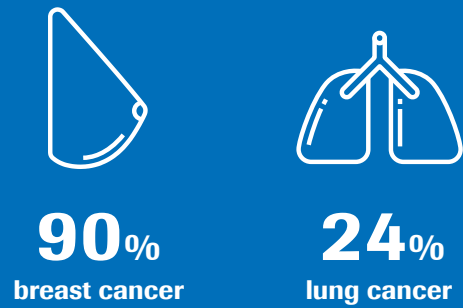

Worldwide data varies from developed v middle income v low income countries, where early detection, adequate diagnosis and treatment greatly varies (Coleman et al., 2008)

With the introduction of targeted therapies that can result in dramatically different outcomes based on subtype, the importance of accurate classification has been amplified.³²

Up to 30% of lung biopsies require adjunct IHC testing after morphologic evaluation.¹⁴ IHC and more specifically, key panels of IHC antibodies, provides the necessary complement in the routine diagnosis and subtyping of lung cancer.³²

Clinical guidelines recommend routine testing for genetic mutations in all adenocarcinomas including ALK EML⁴ gene rearrangement. Testing is recommended immediately after establishing histology and is required prior to initiating targeted therapy for a patient. The current clinical practice and guidelines for ALK testing includes IHC and FISH.⁷

IHC is key in stratifying lung cancer


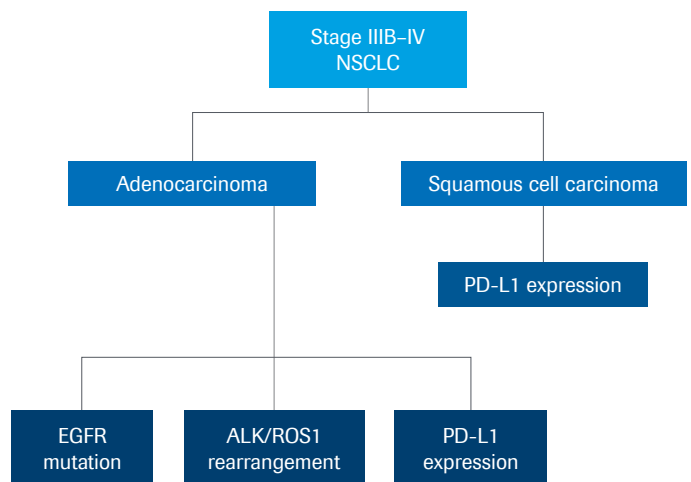


Figure 1. Common testing algorithm to determine indications for appropriate treatment in lung cancer

Treatment options for non-small cell lung cancer

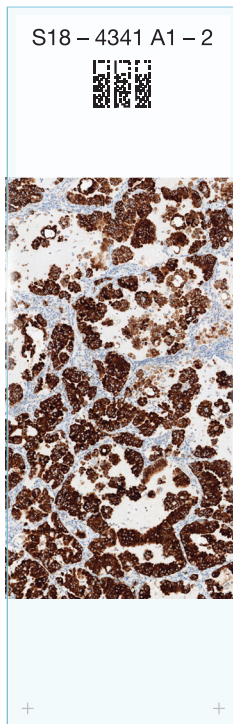

XALKORI, ZYKADIA, ALECENSA and LORBRENA are indicated for the treatment of patients with metastatic NSCLC whose tumors are ALK-positive as detected by an approved testing method for ALK.^{2, 3, 4, 5}

XALKORI (crizotinib) is indicated for the treatment of patients with ALK positive metastatic NSCLC and other kinases.³

ZYKADIA (ceritinib) is indicated for the treatment of patients with ALK-positive metastatic NSCLC who have had no previous treatment, or who have progressed on, or who are intolerant to, crizotinib.⁴

ALECENSA (alectinib) is indicated for the treatment of patients with ALK-positive metastatic NSCLC who have had no previous treatment, or who have progressed on, or who are intolerant to, crizotinib.⁵

LORBRENA (lorlatinib) is indicated for the treatment of patients with anaplastic lymphoma kinase (ALK)-positive metastatic non-small cell lung cancer (NSCLC) whose disease has progressed on crizotinib and at least one other ALK inhibitor for metastatic disease; or alectinib as the first ALK inhibitor therapy for metastatic disease; or ceritinib as the first ALK inhibitor therapy for metastatic disease.²


	PROFILE 1014 trial		ASCEND-4 trial		ALEX trial		CROWN trial	
Progression-Free Survival	XALKORI (n=172)	Chemotherapy (n=171)	ZYKADIA (n=189)	Chemotherapy (n=187)	ALECENSA (n=152)	XALKORI (n=151)	LORBRENA (n=149)	XALKORI (n=147)
Median, months (95% CI) ^[a]	10.9 (8.3, 13.9)	7.0 (6.8, 8.2)	16.6 (12.6, 27.2)	8.1 (5.8, 11.1)	34.8 (17.7-NR)	11.1 (9.1, 13.1)	NR	9.3
HR (95% CI) ^[b]	0.45 (0.35, 0.60)		0.55 (0.42, 0.73)		0.47 (0.34, 0.65)		0.28 (0.19, 0.41)	
p-valued ^[c]	<0.001		<0.0001		<0.0001		<0.001	

Figure 4. Clinical benefit of XALKORI, ZYKADIA and ALECENSA (progression free survival)
 HR=hazard ratio; CI=confidence interval; BIRC=Blinded Independent Review Committee; NR=not reached; NE=not estimable
 [a] Estimated using the Kaplan-Meier method. [b] A Cox regression model stratified by randomization stratification factors (WHO performance status: 0 vs. 1-2; presence or absence of BM, presence or absence of previous neo-/adjuvant chemotherapy) was used to estimate the hazard ratio of PFS, along with 95% CI based on the Wald test. [c] Based on the stratified log-rank test (same stratification as [b]).

Standardization of ALK IHC testing: Specimen type and controls

Patients with late stage lung cancer need a fast, reliable and standardized way to assess treatment options. Roche developed the VENTANA ALK (D5F3) Assay to be used with OptiView DAB IHC Detection and Amp to identify those patients who are eligible for ALK targeted therapy. A full range of human NSCLC tissue specimen types can be tested including resections, needle biopsies, bronchial biopsies and formalin-fixed, paraffin-embedded cell blocks.

A matched negative reagent control slide must be run for every specimen to aid in the interpretation of results. System-level controls must be run with patient samples. They can be either human appendix or known ALK-positive/negative NSCLC tissue samples.

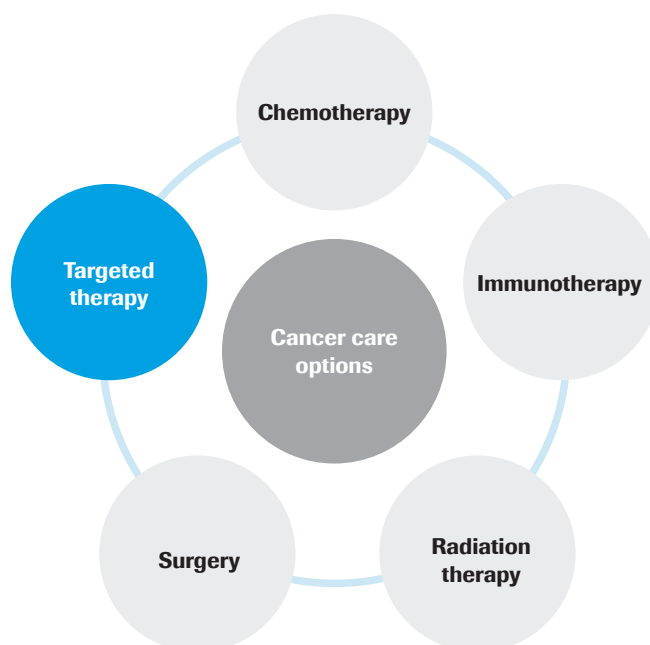

Best in quality

The VENTANA ALK (D5F3) Assay stained with OptiView DAB Detection and Amp scored high in External Quality Assurance testing vs. all other ready-to-use antibodies for demonstration of ALK rearrangement.²⁰

Fast turnaround time

The approved VENTANA ALK (D5F3) Assay stained with OptiView DAB Detection and Amp is a 4 1/2-hour, fully automated test to be stained with all other routine IHC testing for same-day results and to meet current CAP/IASLC/AMP guidelines for testing patients with lung cancer.²⁸

Easy to score

The sensitivity of the approved VENTANA ALK (D5F3) Assay stained with OptiView DAB Detection and Amp enables a reproducible, binary scoring system for evaluating staining results without the need for quantification of cells or staining.²¹

Workflow efficiency

The VENTANA ALK (D5F3) Assay is fully optimized for use on BenchMark IHC/ISH automated staining instruments.



VENTANA ALK (D5F3) Assay vs. FISH

Technical benefits of IHC testing

ALK FISH can present technical challenges in evaluating patient results and offers the potential for false negatives. Recent studies indicate that the VENTANA ALK (D5F3) Assay stained with OptiView DAB Detection and Amp is sensitive and specific for determination of ALK status, and a better alternative to ALK FISH. There are reports of ALK IHC-positive, FISH-negative patients benefiting from treatment with XALKORI.^{22, 23, 24, 25, 26}

One study, found that “Dichotomous ALK-IHC is superior to ALK-FISH on small biopsies and FNA to predict tumor response and survival to crizotinib for advanced NSCLC patients.”²⁷

Figure 5. Comparison of VENTANA ALK (D5F3) Assay stained with OptiView DAB Detection and Amp vs. FISH testing for determining ALK status

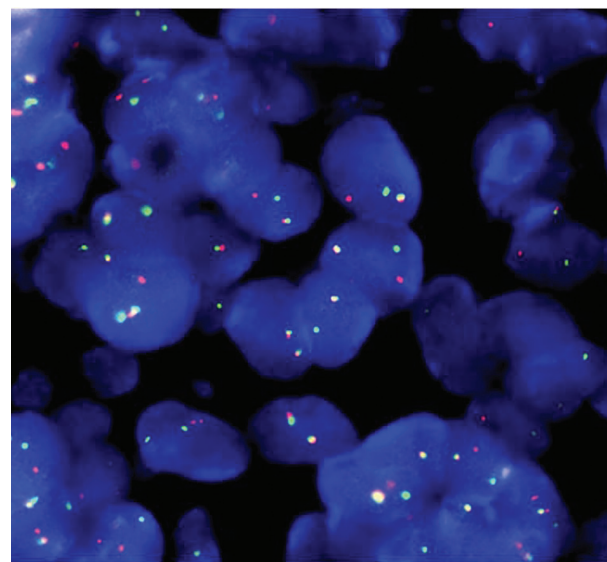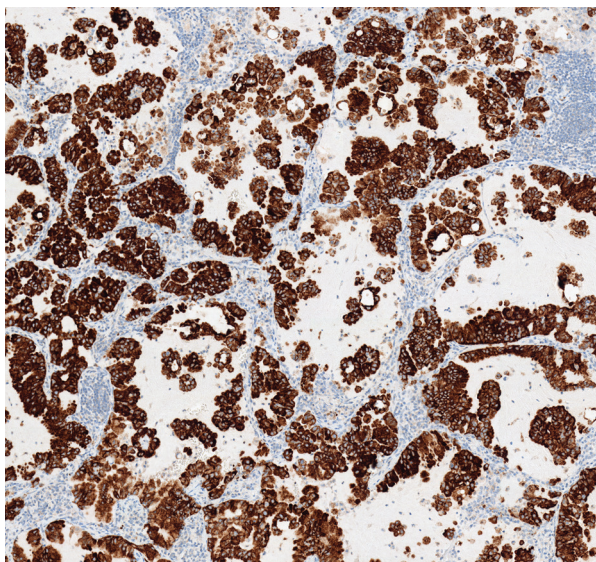

VENTANA ALK (D5F3) Assay with OptiView DAB Detection and Amp

Scoring process

- Binary (+/-) scoring
- Presence of strong granular cytoplasmic staining in tumor cells (any percentage of positive tumor cells)

Turnaround times

- 4 1/2 hours, fully automated
- Routine IHC testing

Brightfield vs. fluorescent staining

- Standard brightfield microscope
- Fully archivable results
- Full visibility of tumor morphology

ALK FISH

Scoring process

- Requires a dual color scoring algorithm
- Requires 50 enumerable cells and specific cutoff ratios to be calculated

Turnaround times


- 12+ hours, semi-automated
- Typically batch or send-out testing

Brightfield vs. fluorescent staining


- Requires a fluorescent microscope
- Staining and signal fade over time
- Loss of tissue morphology

VENTANA ALK (D5F3) Assay vs. other ALK IHC

Figure 6. NSCLC tissue samples stained with different ALK IHC antibodies clones

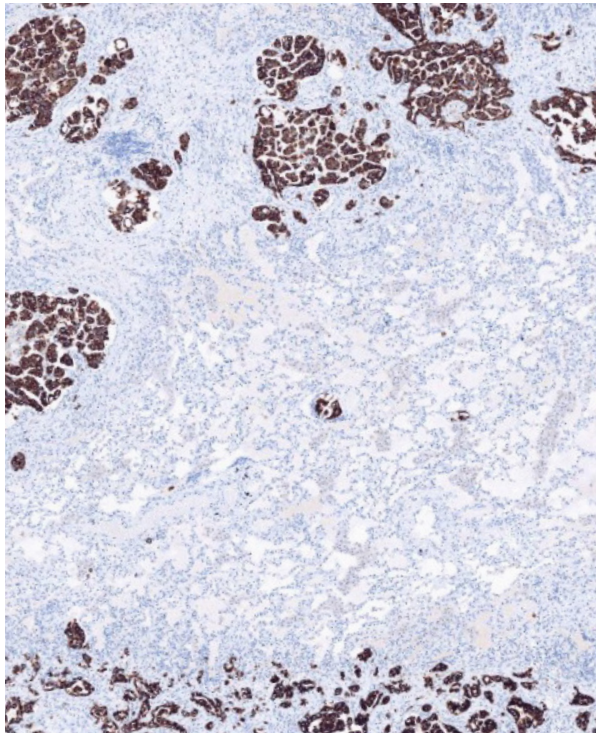

VENTANA ALK (D5F3) Assay with OptiView DAB Detection and Amp

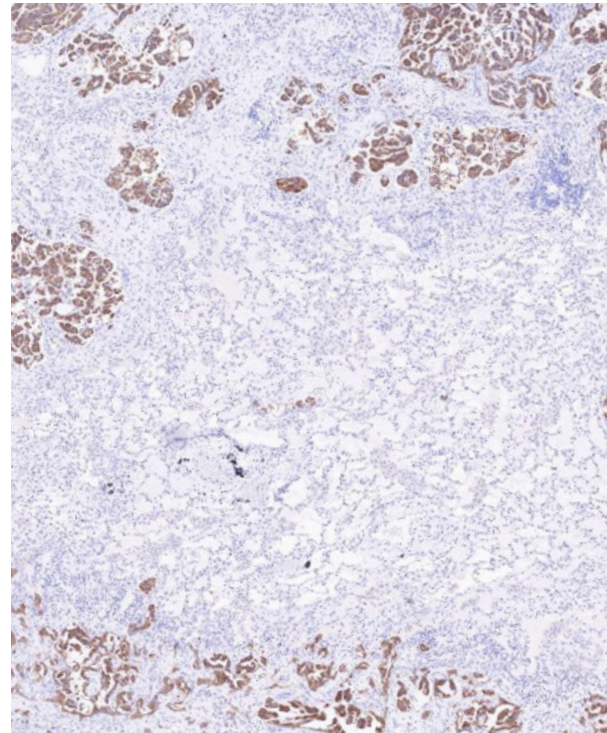

ALK antibody, clone 5A4 with Polymer Detection

ALK antibody clones

The VENTANA ALK (D5F3) Assay stained with OptiView DAB Detection and Amp is the only assay approved as an aid to identify patients eligible for treatment with XALKORI, ZYKADIA, ALECENSA or LORBRENA.²¹

It consistently scores high performance for proficiency and technical External Quality Assessment schemes (CAP, ESP, NordiQC).^{28, 29, 30}

Molecular testing

Current molecular testing does not identify all known ALK genetic variants and is not recommended as an alternative testing method to select patients for ALK inhibitor therapy.⁷



Over ten genetic variants of EML4-ALK mutations have been identified. VENTANA ALK (D5F3) Assay stained with OptiView DAB Detection and Amp identifies a conserved protein sequence common to known variants of the ALK mutation.

- There are multiple genetic variants of the ALK mutation that lead to NSCLC
- Molecular testing does not identify all ALK gene variants and can miss positive cases
- D5F3 clone is specific to the common kinase domain of all ALK mutations and should identify all genetic variants

Molecular testing techniques rely upon good sample integrity and require sophisticated computational analysis to interpret results. Formalin-fixed, paraffin-embedded tissues provide a significant challenge as genetic material is known to degrade in sample preparation. Even when properly performed, interpreting the results of these techniques is not standardized.⁷


ALK testing with VENTANA ALK (D5F3) Assay with OptiView DAB Detection and Amp offers many benefits:

- A fully automated test to select patients for treatment with XALKORI, ZYKADIA, ALECENSA or LORBRENA
- Fastest turnaround time to meet the current CAP/IASLC/AMP guidelines for testing lung patients
- Can be integrated into a routine IHC panel of antibodies to stratify NSCLC patients

Figure 7. Different variants of EML4-ALK and non-EML4 fusion partners⁸

The EML4 genetic sequence is diverse and creates multiple targets for PCR

A conserved protein sequence common to all known ALK mutations is detected by VENTANA ALK (D5F3) Assay

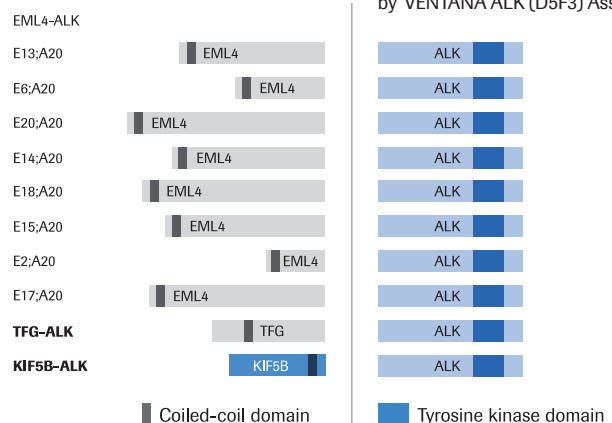

Visit ALKihc.com and contact your local Roche representative to learn more.

Ordering information

Product name	Catalog number	Ordering code	Quantity
VENTANA-ALK (D5F3) CDx Assay	790-4796	06687199001	50 tests
Rabbit Monoclonal Negative Control Ig	790-4795	06683380001	250 tests
OptiView DAB IHC Detection Kit	760-700	06396500001	250 tests
OptiView Amplification Kit	760-099	06396518001	50 tests

References

1. ESMO international guidelines for ALK testing in NSCLC (updated 2018) Planchard et al., *Annals of Oncology*, 2018
2. LORBRENA® (lorlatinib) [package insert]. New York, NY: Pfizer; 2021.
3. XALKORI® (crizotinib) [package insert]. New York, NY: Pfizer; 2012.
4. ZYKADIA (ceritinib) [package insert]. Whippany, NJ: Novartis Pharmaceuticals Corporation 2016.
5. ALECENSA (alectinib) [package insert], San Francisco, CA: Genentech.
6. Ferlay, J., Soerjomataram, I., Ervik, M., Dikshit, R., Eser, S., Mathers, C., Rebelo, M., Parkin, D.M., Forman, D., Bray, F. (2012). GLOBOCAN v1.0, Cancer Incidence and Mortality Worldwide: IARC CancerBase No. 11 Lyon, France: International Agency for Research on Cancer; 2013. Available at: <http://globocan.iarc.fr> (last accessed March 2016).
7. Lindeman NI, Cagle PT, Beasley MB et al. Molecular testing guideline for selection of lung cancer patients for EGFR and ALK tyrosine kinase inhibitors: guideline from the College of American Pathologists, International Association for the Study of Lung Cancer, and Association for Molecular Pathology. *Journal of Thoracic Oncology*. Volume 8(7), July 2013, p 823–859. doi: 10.1097/JTO.0b013e318290868f.
8. Shaw AT, Yeap BY, Mino-Kenudson M, et al. Clinical features and outcome of patients with non-small-cell lung cancer who harbor EML4-ALK. *J Clin Oncol*. 2009;27:4247–4253.
9. NCCN Clinical Practice Guidelines in Oncology.™ Non-small cell lung cancer. v 2.2010. Available at: http://www.nccn.org/professionals/physician_gls/PDF/nscl.pdf Accessed August 1, 2014.
10. <https://ourworldindata.org/>
11. American Cancer Society, October 1, 2019
12. The American Cancer Society medical and editorial content team, last medical review Sep 2019)
13. Tacha D, Yu C, Bremer R, Qi W, Haas T. A 6-antibody panel for the classification of lung adenocarcinoma versus squamous cell carcinoma. *Appl Immunohistochem Mol Morphol*. 2012;20:201–207.
14. Rossi, et al. Subtyping Non-Small Cell Lung Cancer. *Int J Surg Pathol*. 21:326.2013.
15. Capelozzi, V.L. Role of immunohistochemistry in the diagnosis of lung cancer. *J Bras Pneumol*. 2009;35(4):375–382.
16. World Health Organization. International Agency for Research on Cancer. GLOBOCAN 2012: Estimated Cancer Incidence, Mortality and Prevalence Worldwide in 2012. Lyon, France <http://globocan.iarc.fr/Default.aspx>. Accessed August 1, 2014.
17. Lung Cancer Survival Rates and Prognosis. National Cancer Institute at the National Institutes of Health. Bethesda, MD. <http://www.cancer.gov/cancertopics/types/lung/cancer-survival-prognosis> Accessed August 1, 2014.
18. Dr William Pao MD, Nicolas Girard MD. New driver mutations in non-small-cell testslung cancer *The Lancet Oncology* 1 February 2011 (Vol. 12, Issue 2, 175–180) doi: 10.1016/S1470-2045(10)70087-5.
19. Manabu Soda, Young Lim Choi, Munehiro Enomoto, et al. Identification of the transforming EML4-ALK fusion gene in non-small-cell lung cancer. *Nature*. August 2, 2007;448(7153):561–566.
20. http://www.nordiqc.org/downloads/assessments/45_14.pdf.
21. Package Insert [VENTANA anti-ALK (D5F3) Rabbit Monoclonal Primary Antibody] Available online: <https://pim-eservices.roche.com>, and entering ordering code 06679072001. Interpretation Guide for ALK (D5F3) in Non-Small Cell Lung Carcinoma (NSCLC) Available online: <https://pim-eservices.roche.com>, and entering ordering code 06679072001
22. Zhou J, Zhao J, Sun K, Wang B, Wang L, et al. Accurate and Economical Detection of ALK Positive Lung Adenocarcinoma with Semiquantitative Immunohistochemical Screening. *PLoS ONE* (2014) 9(3): e92828. doi:10.1371/journal.pone.0092828.
23. Ling Shan, Fang Lian, Lei Guo, Xin Yang, Jianming Ying and Dongmei Lin. Combination of conventional immunohistochemistry and qRT-PCR to detect ALK rearrangement. *Diagnostic Pathology* 2014, 9:3. doi:10.1186/1746-1596-9-3 .
24. Ying, J.; Guo, L.; Qiu, T.; Shan, L.; Ling, Y.; Liu, X.; Lu, N. Diagnostic value of a novel fully automated immunochemistry assay for detection of ALK rearrangement in primary lung adenocarcinoma. *Annals of Oncology*. 24(10):2589–2593, October 2013. *Modern Pathology* (2013) 26, 1545–1553; doi:10.1038/modpathol.2013.87; published online 7 June 2013.
25. Selinger CI1, Rogers TM, Russell PA, et al. Testing for ALK rearrangement in lung adenocarcinoma: a multicenter comparison of immunohistochemistry and fluorescent in situ hybridization. *Modern Pathology* (2013) 26, 1545–1553; doi:10.1038/modpathol.2013.87; published online 7 June 2013.
26. Greta Ali, Agnese Proietti, Serena Pelliccioni, et al. ALK Rearrangement in a Large Series of Consecutive Non-Small Cell Lung Cancers: Comparison Between a New Immunohistochemical Approach and Fluorescent In Situ Hybridization for the Screening of Patients Eligible for Crizotinib Treatment. *Arch Pathol Lab Med*. 2014 June 2.
27. van der Wekken et al Dichotomous ALK-IHC is a better predictor for ALK inhibition outcome than traditional ALK-FISH in advanced non-small cell lung cancer, *Clinical Cancer Research*, 2017. Available at <http://clincancerres.aacrjournals.org/content/early/2017/02/09/1078-0432.CCR-16-1631>.
28. College of American Pathologists (2017-A).
29. European Society of Pathology (2017) Lung External Quality Assessment Run 2017
30. NordiQC (2017) NordiQC External Quality Assurance in Immunohistochemistry run 51
31. Tacha D, Yu C, Bremer R, Qi W, Haas T. A 6-antibody panel for the classification of lung adenocarcinoma versus squamous cell carcinoma. *Appl Immunohistochem Mol Morphol*. 2012;20:2
32. apelozi, V.L. Role of immunohistochemistry in the diagnosis of lung cancer. *J Bras Pneumol*. 2009;35(4):3
33. NCCN Clinical Practice Guidelines in Oncology (NCCN Guidelines®). "Non-small Cell Lung Cancer" Version 4.2021-March3,2021. Available at www.nccn.org/patients. Accessed March 24, 2021

BENCHMARK, OPTIVIEW and VENTANA are trademarks of Roche. All other trademarks are the property of their respective owners.

© 2021 Roche. MC-US-04325-0321

Roche Diagnostics
9115 Hague Road
Indianapolis, IN 46250

diagnostics.roche.com