




# Vaginal Swab Specimen Collection Guide

## Clinician-collection

with the **cobas®** PCR Media Uni Swab Sample Kit

### Note

**Do not** pre-wet the swab in **cobas®** PCR Media before collection.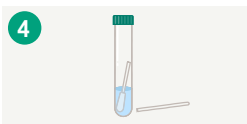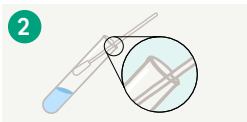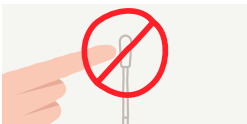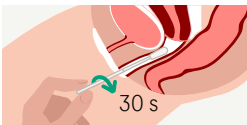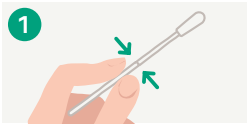

### Collect

- To collect the specimen, hold the woven swab with the scoreline above your hand and insert the swab about 5 cm (2 inches) into the vaginal opening.
- Gently turn the swab for about 30 seconds while rubbing the swab against the walls of the vagina.
- Withdraw the swab carefully.
- Do not let the swab touch any surface before placing it into the collection tube.

### Align

- Remove the cap from the **cobas®** PCR Media Tube.
- Lower the swab specimen into the tube until the visible scoreline on the swab shaft is aligned with the tube rim.
- The bud of the swab should not be submerged into the liquid prior to breaking the shaft.

### Break


- Carefully leverage the swab against the tube rim to break the swab shaft at the scoreline.

### Close

- Tightly close the **cobas®** PCR Media Tube. The specimen is now ready for transport.
- Discard the top portion of the swab.

### Procedural notes

- Vaginal lubricants, speculum jellies, creams, and gels containing carbomer(s) may interfere with the test and should not be used during or prior to sample collection.
- If the collected specimen contains excess blood (specimen has a red or brown color), it should be discarded and not used for testing.
- Avoid contact of the **cobas®** PCR Media with the skin, eyes or mucous membranes. If contact does occur, immediately wash with large amounts of water.

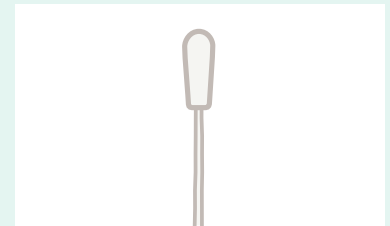Woven swab

### Specimen transport and storage

- Transport and store the **cobas®** PCR Media Tube containing the collection swab at 2°C to 30°C.
- The specimen should only contain 1 woven swab and may be rejected if the tube contains no swab or 2 swabs.

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