Vaginal Swab Specimen Collection Guide

Self-collection in a clinical setting


with the **cobas®** PCR Media Uni Swab Sample Kit

Preparing for sample collection

- Wash hands prior to collection. Undress to expose the vaginal area. Put yourself in a comfortable position.
- Remove the collection tube and the woven swab from the collection kit.


Note

Do not pre-wet the swab in **cobas®** PCR Media before collection.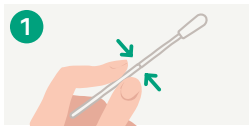


Position

- In one hand, hold the woven swab with the scoreline above your hand and with the other hand separate the folds of skin around the vaginal opening (labia).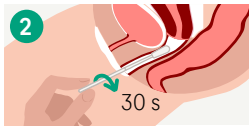


Collect

- Insert the swab about 5 cm (2 inches) into the vaginal opening.
- Gently turn the swab for about 30 seconds while rubbing the swab against the wall of the vagina.
- Remove the swab carefully.
- Do not touch the swab to any surface before placing it into the collection tube.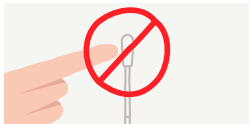

Open tube

- While holding the swab in the same hand, remove the cap from the tube as shown in the diagram.

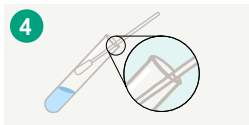Align

- Lower the swab into the tube until the visible scoreline on the swab shaft is lined up with the tube rim.

Break

- Carefully lean the swab against the tube rim to break the swab shaft at the scoreline.


Close

- Tightly close the **cobas®** PCR Media Tube.
- Return the sample to your healthcare provider as instructed.
- Discard the top portion of the swab.

Handling precautions

- Handle the collection tube carefully, as tube media can cause irritation if contacted with skin or other body parts.
- If the tube contents are spilled on your skin, wash the affected area with soap and water. If contents splash into your eyes, flush them with water immediately.
- Do not attempt to clean up any spilled contents of the tube.
- If any of these events occur, always notify your healthcare provider.

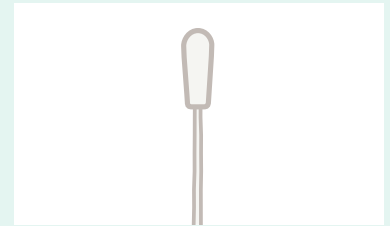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