ENGLISH

Manufactured by:

LumiraDx Inc, 221 Crescent Street, Waltham MA 02453 USA

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ART-00040 Rev9

The LumiraDx Connect Hub

The Connect Hub is a hardware product that plugs directly into a power supply and synchronizes data between LumiraDx Instrument(s) and LumiraDx Connect Servers.

The Connect Hub supports power outlets for most regions.

A green "Power indicator" on the front panel shows the Connect Hub is powered on and properly connected to the mains.

The Connect Hub connects to the internet through either a wired Ethernet connection or a Wi-Fi connection. The only set-up required is to get the Connect Hub connected to the internet.

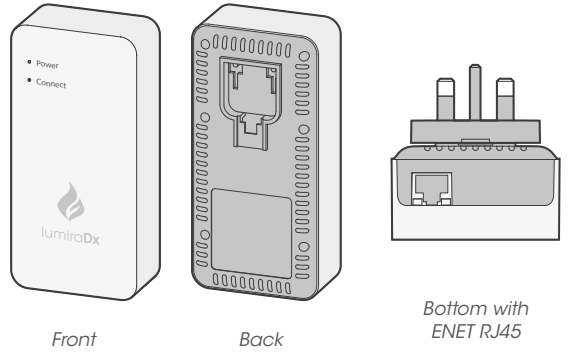
Please see the Connect User Manual on our website www.lumiradx.com for terms of use and instructions on getting your Connect Hub connected.

Once communicating to the network, an automatic connection occurs to the appropriate server and instruments which are within its range. The "Connect" indicator switches from flashing green to solid green once successful communications are made.


The LumiraDx Connect Hub - Contents of the Box

The contents of the Connect Hub package include the following:

- The LumiraDx Connect Hub

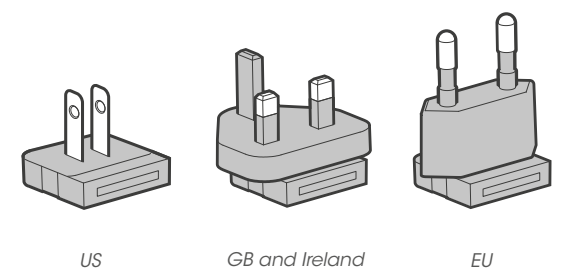- Three LumiraDx Connect Hub Power Blades - used to plug the Connect Hub into the power supply:

OS-04271-01 APPROVED FOR USE IN THE US.

EMERGENCY USE AUTHORIZED ONLY.

Symbols Glossary

	Equipment for Indoor Use Only
	Double Insulated Equipment
	Refer to Instructions
	The CE mark indicates that the product complies with all relevant European Directives.

Safety Notices

	Warning: Do not use this product near water.
	Warning: There are no serviceable parts in this product. Do not open.
	Warning: Do not wipe with alcohol. Use a damp cloth with mild detergent that is safe for plastics.
	Warning: Do not use this device near sources of strong electromagnetic radiation as these can interfere with the proper operation.
	Vents around the device should not be blocked, place the device in a well-ventilated area and out of direct sunlight.

Environmental Practices

LumiraDx complies with all environmental legislation in each country where the product is sold. Please refer to <http://www.lumiradx.com> for further information.

WEEE Information

European Directive 2012/19/EU requires that the equipment bearing this symbol on the product and/or its packaging must not be disposed of with unsorted waste. The product should be recycled in accordance with local regulations. This product does not contain batteries.

Changes or modifications not expressly approved by LumiraDx UK Ltd could void the user's authority to operate the Instrument.

Compliance Notice

LumiraDx Inc hereby declares that the product complies with all relevant European Directives and Regulations at the time of placing on the market. The EC Declaration of Conformity may be found at <http://www.lumiradx.com>

The following is information regarding radio equipment intentionally emitting radio waves:

(a) The radio equipment operates in 2.4 GHz ISM band between 2400 Mhz to 2483.5 Mhz. The maximum power transmitted in this frequency band is 20 dBm (e.i.r.p).

(b) The radio equipment can also operate in the 5 GHz band between 5150 Mhz to 5350 Mhz and 5450 Mhz to 5725 Mhz. The maximum power transmitted in this frequency band is 23 dBm (e.i.r.p).

ANATEL - Brazil. This equipment is not entitled to protection against harmful interference and cannot cause interference in duly authorized systems. For more information, access the ANATEL website at www.anatel.gov.br.

Details of relevant Intellectual Property regarding this product can be found at www.lumiradx.com/IP

Contains FCCID: 2ABCB-RPI3BP

Legal Notices:

Warranty - 2 years from date of purchase.

Copyright © 2021 LumiraDx Group Limited. All rights reserved, worldwide.

LumiraDx and Flame logo are protected trademarks of LumiraDx International LTD. Full details of these and other registrations of LumiraDx can be found at lumiradx.com/IP. All other trademarks are the property of their respective owners.

CE CE mark applies to LumiraDx Instrument, Test Strips, Quality Controls and Connect Hub only.

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Copyright © 2021 LumiraDx Group Limited. Tous droits réservés à l'échelle mondiale.
Copyright © 2021 LumiraDx Group Limited. Alle Rechte weltweit vorbehalten.