



Quick Start Guide for the LumiraDx Printer

The LumiraDx Printer can be used to print a Patient or Quality Control (QC) test result onto adhesive paper.

Patient and QC test results can be printed from the Instrument result page following a test, or from the test details page in the Instrument Result History.

Fully charge the batteries for 16 hours before first use.



1. What's in the Box



LumiraDx Printer



USB Micro-B Cable



**4x Single Cell
rechargeable batteries
(1.2V NiMH, AA Cells)**



**9V/1A Power Supply
(Includes Charger Body
and Plug Head)**

2. Setting up the LumiraDx Printer

- 2.1** The Printer comes with a roll of adhesive paper already installed.

Ensure that the paper passes through the paper exit slot.



- 2.2** Turn the Printer over to see the underside.

Remove the sticker over the battery compartment.

Unclip and lift off the battery cover.

Insert the supplied rechargeable batteries into the Printer paying attention to the '+' and '-', then replace the cover.



- 2.3** Connect the USB cable and power cable to the Printer.



- 2.4 Fully charge the batteries for 16 hours** before first use by connecting the Printer to a power source.

- 2.5** Insert the USB connector into one of the USB ports at the rear of the Instrument and press the Printer power button.



3. Replacing the batteries

- 3.1** Unclip and lift off the battery cover.
- 3.2** If there are already batteries in the Printer, remove the batteries and insert 4 fully-charged batteries paying attention to the '+' and '-'. If the batteries are not fully charged, charge for 16 hours before use. Only use 1.2V NiMH AA rechargeable batteries in the Printer.
- 3.3** Align the cover into place and click shut.

4. Replacing the adhesive paper

- 4.1** Pull the clear central lever on the clear paper cover upwards and forwards until it is release from its locked position. To avoid damage do not use excessive force.
- 4.2** Remove the spool and any remaining adhesive paper. Place a new adhesive paper roll into the paper reservoir, holding approx. 2 inches of the paper outside of the Printer. Ensure the new paper is feeding from the bottom of the roll. Close the cover by applying equal amounts of pressure on each side of the cover until the lid is in the locked position.
- 4.3** Now tear the surplus paper across the serrated edge.

5. Warnings and Precautions

- Take care when transporting to prevent damaging the Printer and batteries.
- Keep away from fire and heating sources.
- Avoid batteries contacting with oxidizers and acid.
- Do not expose the batteries to excessive physical shock or vibration.
- Do not short-circuit.
- Keep batteries out of the reach of children.
- Only use 1.2V NiMH AA cell rechargeable batteries.



6. LumiraDx Printer status light

LED Indication	Condition	Solution
On	Printer On	-
Off	Printer Off	-
* * *	Paper Out	Fit new paper
** ** **	Thermal head too hot	Allow head to cool
*** ** *	Battery cut-out (no charge remaining)	Recharge batteries
**** *****	Battery low (approx. 20% charge remaining)	Recharge batteries

7. Supplies from LumiraDx

Supplies can be obtained by contacting LumiraDx Customer Services at customerservices@lumiradx.com.

Item	Order Code
Adhesive paper rolls	L002120101006

8. Customer Services

Please email or call Customer Services if you have questions about this product.

Email: customerservices@lumiradx.com

Phone: for your local Customer Services phone number please visit the Contact Us page at lumiradx.com

Manufacturer Information

Martel Instruments Ltd, Unit 9 Stanelaw Way, Tanfield Lea Ind Est South, Stanley, DH9 9XG, UK

Copyright © 2025 LumiraDx and affiliates. All rights reserved. LumiraDx and Flame logo are protected trademarks of LumiraDx International LTD. Full details of these and other registrations of LumiraDx can be found at lumiradx.com/IP. All other trademarks are the property of their respective owners.

ART-03307 R2